

PRODUCT KEY


A: 69099 	E: 72294-L 
B: 73000-SA 	F: 55009 
C: 72280 	G: 55013-IL  Other tongue blade options include: *55013-IL, 55014-IL, 55015-IL, 55016-IL, 55017-IL, 55018-IL
D: 55012 	H: 72284 



Attach the Rail Clamp (A) to the table rail and insert the vertical post of the StrongArm® (B), with the flat side facing the flat side of the Rail Clamp.




Attach the Clasper (C) into the Quick Connect fitting of the StrongArm. Attach the clasper (C) onto the ring (H), close to the knobs (I), on the same side of patient's maxilla side. The knobs on the ring need to align with the corners of the patient's mouth. Angle the mandible side of the ring upwards, slightly, and loosely tighten the ring to hold shape.

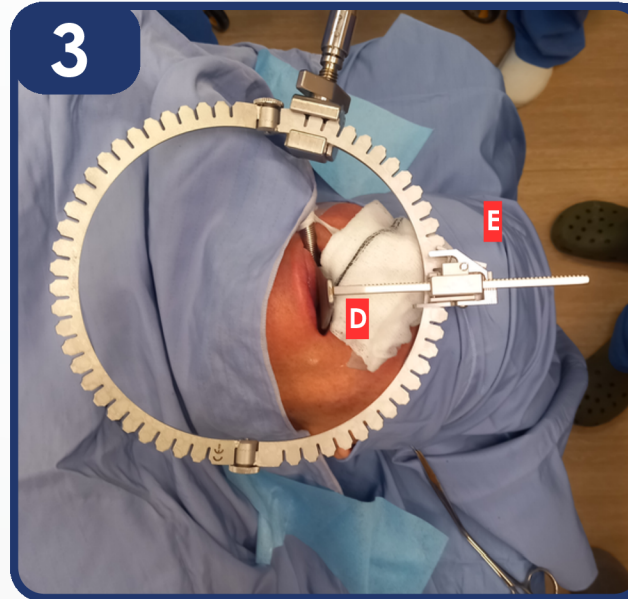

After protecting the teeth/gums – insert the maxilla blade (D) into the mouth and secure it with a tilt ratchet (E) onto the ring. Once attached, retract slightly to stabilize the ring setup.

4

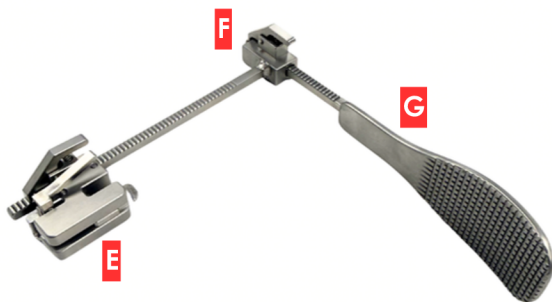

Attach the Tongue Blade (G) into the extended ratchet (F). Insert the extended ratchet into the tilt ratchet (E).

5



Attach the Tongue Blade (G) into the mouth and fixate the tilt ratchet (F) onto the ring. Control the depth of the blade by moving it up and down on the extended ratchet. At the preferred depth, begin retracting the tongue and lower jaw.

6



Optimize the field of view as needed. Best use of the system is achieved when the angle of the ring is adjusted so as to be as close to the patient, without touching them, as possible. When all is set, lock the segmented knobs on the ring and ensure the strong arm is properly tightened.

7



Follow the manufacturer's instructions for proper robot setup to avoid any collision with ENTrac®.

To break down the system, use this setup guide in reverse.

